VERMEIREN

Streeters

INSTRUCTION MANUAL
MODE D'EMPLOI
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
ISTRUZIONI PER L'USO
MANUAL DE INSTRUCCIONES
INSTRUKCJA OBSŁUGI

Instructions to the specialist dealer

This instruction manual is part and parcel of the product and must accompany every product sold.

Version: D, 2017-01

All rights reserved, including translation.

No part of this manual may be reproduced in any form what so ever (print, photocopy, microfilm or any other process) without written permission of the publisher, or processed, duplicated or distributed by using electronic systems.

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© Vermeiren GROUP 2017

INSTRUCTION MANUAL Streeters

First of all we wish to thank you for the trust you placed in us by selecting a VERMEIREN product.

Before using the Streeter, read the instruction manual carefully: it will familiarize you with this product. You will also be given hints about the care and service of your Streeter. Please remember that observance of our instructions would keep your Streeter in an excellent condition of preservation and it will function perfectly even after years of use.

If you have any further questions, kindly consult your specialist dealer.

Technical details

Red Model

Height: 460-570 mm
Length: 740 mm
Folded: 200 mm
Width: 530 mm
Waist: 340 mm
Weight: 4.75kg
Max. user weight: 50kg

Beige Model

Height: 470-600 mm
Length: 900 mm
Folded: 170 mm
Width: 550 mm
Waist: 360 mm
Weight: 5 kg
Max. user weight: 65kg

Green Model

Height: 600-730 mm
Length: 960 mm
Folded: 210 mm
Width: 590 mm
Waist: 400 mm
Weight: 5.75 kg
Max. user weight: 80kg

Blue Model

Height: 720-900 mm
Length: 1020 mm
Folded: 210 mm
Width: 630 mm
Waist: 430 mm
Weight: 6.45 kg
Max. user weight: 90kg

Scope of delivery

- Lower frame (incl. 4 wheels)
- Instruction manual

Assembling

1. Press on the spring button, until the leg is unlocked. Adjust the leg to the desired length and check the correct position of the spring button. Adjust all legs at the same height.

2. Check that the walker is locked in the unfolded position, before using the walker. Therefore grasp the sides of the walker and try to close the walker. Use the Streeter only on flat, solid underground. The use on stairs is prohibited.

Safety

➔ Do not lean against or on the Streeter -- danger of tipping over !

➔ Do not convey persons or objects -- danger of injury !

➔ The Streeter must only be set up indoors on level surfaces. When in use, check that all 4 castors press evenly on the floor.

➔ Repair work may only be undertaken by the specialist dealer.

➔ Take note of the instructions for care and service. The manufacturer is not liable for damage caused by improper servicing/care.

Care

It is best to use a soapy solution for cleaning. The cleaning liquid should in any case have a pH value of at least 6. Do not use a cleaning agent which contains solvents. Observe the instructions of the cleaning liquid used. Kindly use a damp cloth and lukewarm water for cleaning.

Avoid soaking. Do not use metal brushes or any other sharp-edged or pointed cleaning utensils. In the case of damage to the varnish or the woodwork, consult your specialist dealer. Regular care would ensure that your Streeter is preserved in a perfectly functional condition.

Disinfection

For sterilization use only disinfectants suitable for treating varnished metal and wood surfaces. Observe the instructions of the cleaning liquid used. Use only proved processes (for scrubbing disinfection) and disinfectants included in the list of the Robert Koch Institute (information can be obtained from:

www.rki.de). Disinfections may only be undertaken by trained persons. Consult your specialist dealer.

Repairs / Service / Inspection

You should use only genuine VERMEIREN replacement parts. Repairs and renovations may only be undertaken by trained persons. Make use of the services offered by your specialist dealer. He would gladly assist you further in regard to service and repairs.

Before you resume use of the Streeter, it should be serviced according to the following criteria:

- Completeness
- Adjustment functions
- Check the castors (stability, tracking, fixing screws)
- Frames, (deformation, stability)
- Trays, (closed surfaces, splinters, tears)
- damaged varnish (remove possible rust and relacquar)

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ENGLISH

CONTRACTUAL WARRANTY

We offer 5 years of warranty on standard wheelchairs, lightweight wheelchairs 4 years. Electronic wheelchairs, tricycles and other products: 2 years (batteries 6 months) and multiposition wheelchairs 3 years. This warranty is limited to the replacement of defective or spare parts.

APPLICATION CONDITIONS

In order to claim its warranty, part "B" of this card has to be given to your official Vermeiren dealer. The warranty is only valid when parts are replaced by Vermeiren in Belgium.

EXCEPTIONS

This warranty is not valid in case of:

- damage due to incorrect usage of the wheelchair,
- damage during transport,
- involvement in an accident,
- a dismount, modification or repair carried outside of our company and/or official Vermeiren dealership,
- normal wear of the wheelchair,
- non-return of the warranty card

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- please return within 8 days of date of purchase or register your product at our website, <http://www.vermeiren.be/registration>

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VERMEIREN GROUP
Vermeirenplein 1/15
B-2920 Kalmarhout
BELGIUM

Notes

A series of horizontal dotted lines for writing notes.

Notes

A series of horizontal dotted lines for taking notes.

Belgium

N.V. Vermeiren N.V.

Vermeirenplein 1 / 15
B-2920 Kalmthout
Fax: +32(0)3 666 48 94
website: www.vermeiren.be
e-mail: info@vermeiren.be

France

Vermeiren France S.A.

Z. I., 5, Rue d'Ennevelin
F-59710 Avelin
Tel: +33(0)3 28 55 07 98
Fax: +33(0)3 20 90 28 89
website: www.vermeiren.fr
e-mail: info@vermeiren.fr

Italy

Vermeiren Italia

Viale delle Industrie 5
I-20020 Arese MI
Tel: +39 02 99 77 07
Fax: +39 02 93 58 56 17
website: www.vermeiren.it
e-mail: info@vermeiren.it

Poland

Vermeiren Polska Sp. z o.o

ul. Łączna 1
PL-55-100 Trzebnica
Tel: +48(0)71 387 42 00
Fax: +48(0)71 387 05 74
website: www.vermeiren.pl
e-mail: info@vermeiren.pl

Czech Republic

Vermeiren ČR S.R.O.

Nádražní 132
702 00 Ostrava 1
Tel: +420 596 133 923
Fax: +420 596 121 976
website: www.vermeiren.cz
e-mail: info@vermeiren.cz

Germany

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Wahlerstraße 12 a
D-40472 Düsseldorf
Tel: +49(0)211 94 27 90
Fax: +49(0)211 65 36 00
website: www.vermeiren.de
e-mail: info@vermeiren.de

Austria

Vermeiren Austria GmbH

Schärddinger Strasse 4
A-4061 Pasching
Tel: +43(0)7229 64900
Fax: +43(0)7229 64900-90
website: www.vermeiren.at
e-mail: info@vermeiren.at

Switzerland

Vermeiren Suisse S.A.

Eisenbahnstrasse 62
3645 Gwatt (Thun)
Tel: +41(0)33 335 14 75
Fax: +41(0)33 335 14 67
website: www.vermeiren.ch
e-mail: info@vermeiren.ch

Spain / Portugal

Vermeiren Iberica, S.L.

Carratera de Cartellà, Km 0,5
Sant Gregori Parc Industrial Edifici A
17150 Sant Gregori (Girona)
Tel: +34 972 42 84 33
Fax: +34 972 40 50 54
website: www.vermeiren.es
e-mail: info@vermeiren.es

Manufacturer

Vermeiren GROUP

Vermeirenplein 1 / 15
B-2920 Kalmthout
Tel: +32(0)3 620 20 20
Fax: +32(0)3 666 48 94
website: www.vermeiren.be
e-mail: info@vermeiren.be