


Detail A
Scale: 1:3

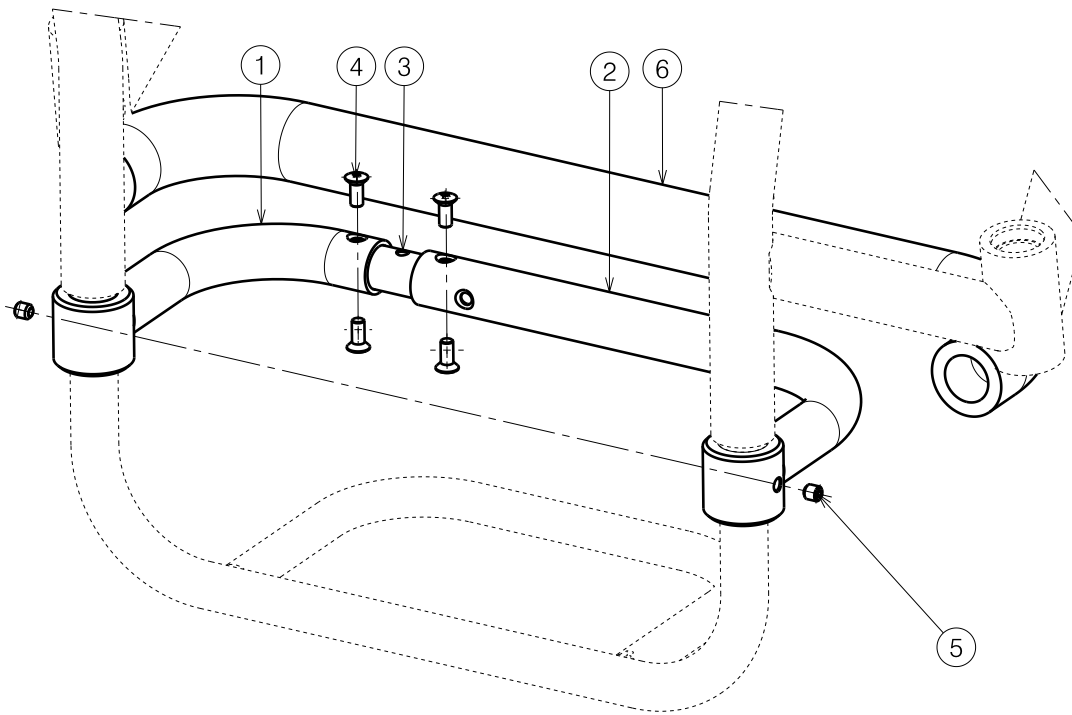

Material			
No.	Plan	Article	Quantity
1	YP02802	28162	1
2	YP02738	28097 (34-48)	1
	YP02737	28098 (24-32)	1
3	XP00720	GP00080	1
4	M5x12	2990933	4
5	M6x6	1998138	2
6	---	1991107	1

STRENGTHENING FOOTREST SAGITTA				Plan Number
				EP00026/B
Scale: 1:3	Drafted	Verified	Confirmed	
Date	10/02/2011	08/06/2011	08/06/2011	
Initials	RM	ACH	TK	