
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


*27 Glue with Weicon AN 302-43 (H900009)
 *28 Lijmen met Weicon AN 302-43 (H900009)

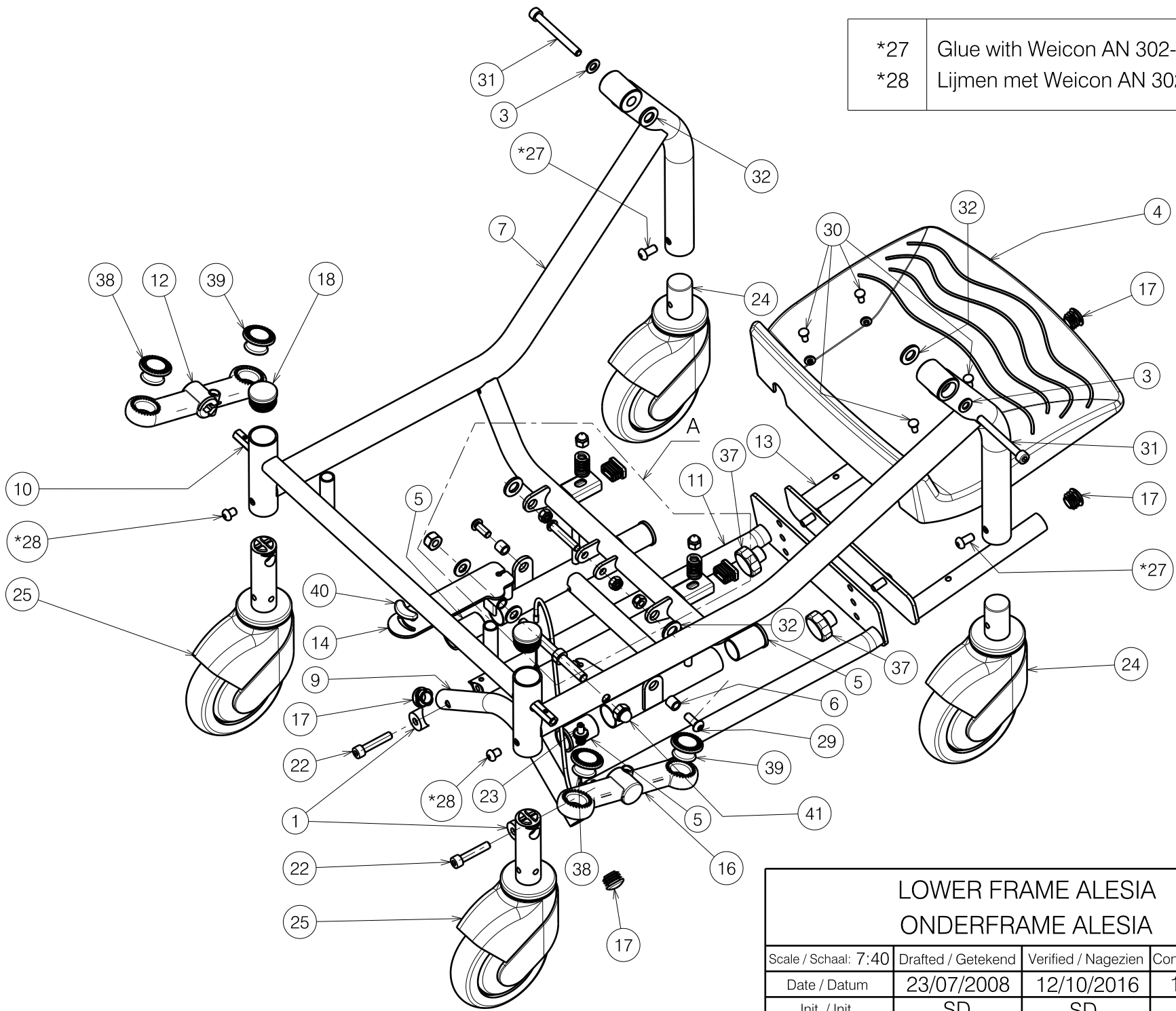


| | | | |
|---|--------------------|---------------------|-----------------------|
| LOWER FRAME ALESIA ONDERFRAME ALESIA | | | |
| Scale / Schaal: 7:40 | Drafted / Getekend | Verified / Nagezien | Confirmed / Bevestigd |
| Date / Datum | 23/07/2008 | 12/10/2016 | 12/10/2016 |
| Init. / Init. | SD | SD | PJ |



| | |
|--------------------|--|
| Plan No. / Plannr. | |
| E000471/L | |
| | |

Material / Materiaal

| No. / Nr. | Plan | Article / Artikel | Number / Aantal |
|-----------|---------|-------------------|-----------------|
| 1 | X000862 | 1006463 | 2 |
| 2 | X001016 | 1001682 | 3 |
| 3 | X001119 | 1001554 | 2 |
| 4 | X004395 | 1902815 | 1 |
| 5 | X004037 | 1902690 | 4 |
| 6 | X004078 | 1699300 | 2 |
| 7 | Y006626 | 1690574 | 1 |
| 8 | Y006980 | 1690674 | 1 |
| 9 | Y007015 | 1691101 | 1 |
| 10 | Y008570 | 1691400 | 1 |
| 11 | Y021868 | 2023501 | 1 |
| 12 | --- | 1912500 | 1 |
| 13 | Y021871 | 2023401 | 1 |
| 14 | Y013418 | 1691074 | 1 |
| 15 | --- | 1001557 | 2 |
| 16 | --- | 1912501 | 1 |
| 17 | --- | 1990648 | 4 |
| 18 | --- | 1001610 | 2 |
| 19 | --- | 1001616 | 2 |
| 22 | --- | 1001500 | 2 |
| 23 | --- | 1900008 | 1 |
| 24 | --- | 1902149 | 2 |
| 25 | --- | 1902150 | 2 |
| 26 | --- | 1902024 | 2 |
| 27 | --- | 1991858 | 2 |
| 28 | --- | 1992177 | 2 |
| 29 | --- | 1992349 | 2 |
| 30 | --- | 1992835 | 4 |
| 31 | --- | 1992973 | 2 |
| 32 | --- | 1993362 | 4 |
| 33 | --- | 1994473 | 1 |
| 34 | X004393 | 1700100 | 1 |
| 35 | --- | 1902846 | 2 |
| 36 | X004267 | 1903027 | 1 |
| 37 | --- | 1998550 | 2 |
| 38 | --- | 1911604 | 2 |
| 39 | --- | 1912499 | 2 |
| 40 | X004295 | 1903132 | 1 |
| 41 | --- | 1009994 | 1 |
| 43 | --- | 1001683 | 1 |

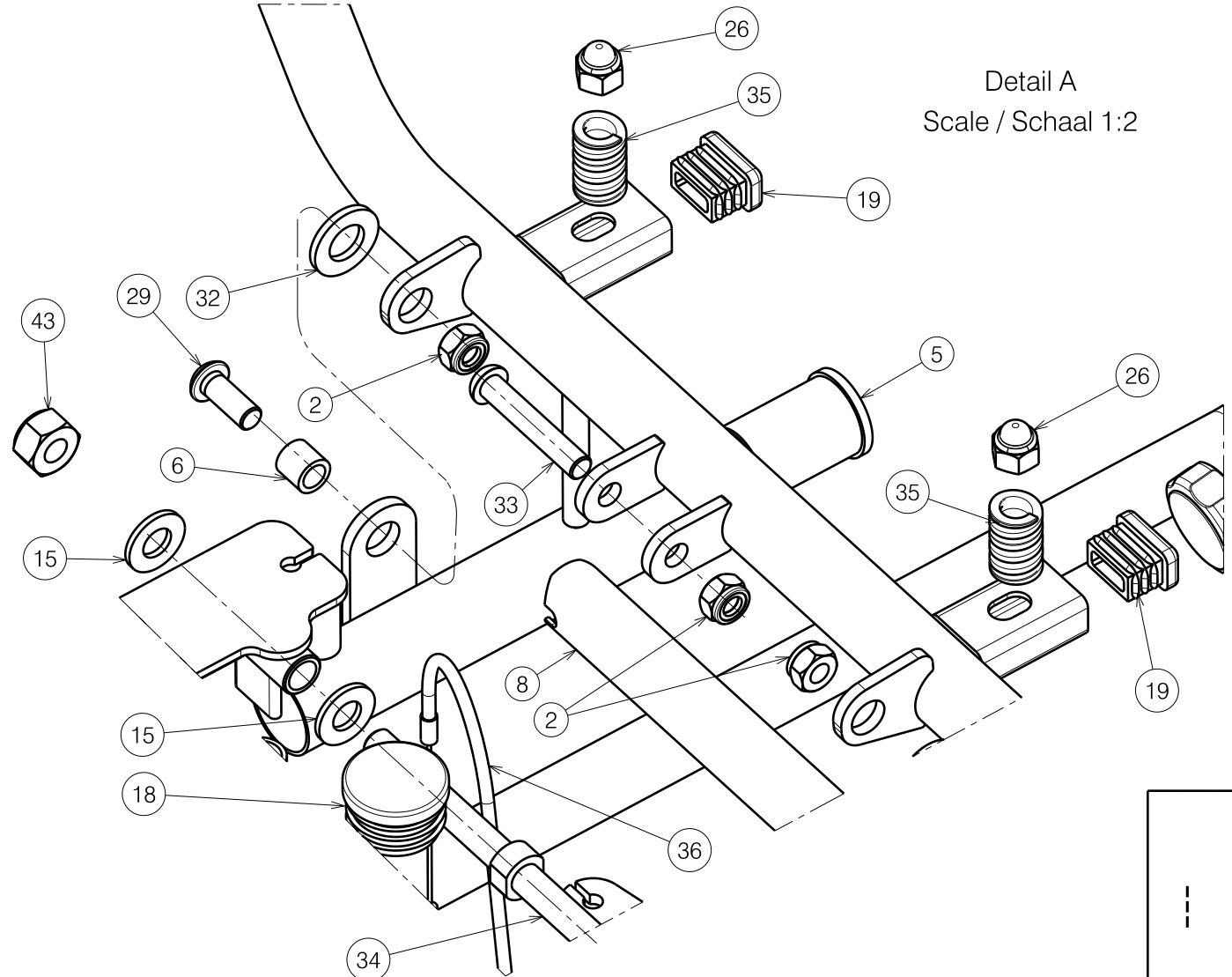

Detail A
Scale / Schaal 1:2

| | | | | |
|---|--------------------|---------------------|-----------------------|--------------------|
| LOWER FRAME ALESIA ONDERFRAME ALESIA | | | | Plan No. / Plannr. |
| | | | | E000471/L |
| Scale / Schaal: 7:40 | Drafted / Getekend | Verified / Nagezien | Confirmed / Bevestigd | |
| Date / Datum | 23/07/2008 | 12/10/2016 | 12/10/2016 | |
| Init. / Init. | SD | SD | PJ | |