



No.	Plan	Article	Qty
1	XC00441	1C520	1
2	3994561	3994561	1
3	3990605	3990605	1
4	3990604	3990604	1
5	YC00680	1C27900	1
6	YC00681	1C28000	4
7	YC00682	1C28100	1
8	XC00179	3990713	1
9	3994560	3994560	1
10	D3.9x9.2 DIN7982	3991105	7
11	M4x25 DIN7985	3991134	5

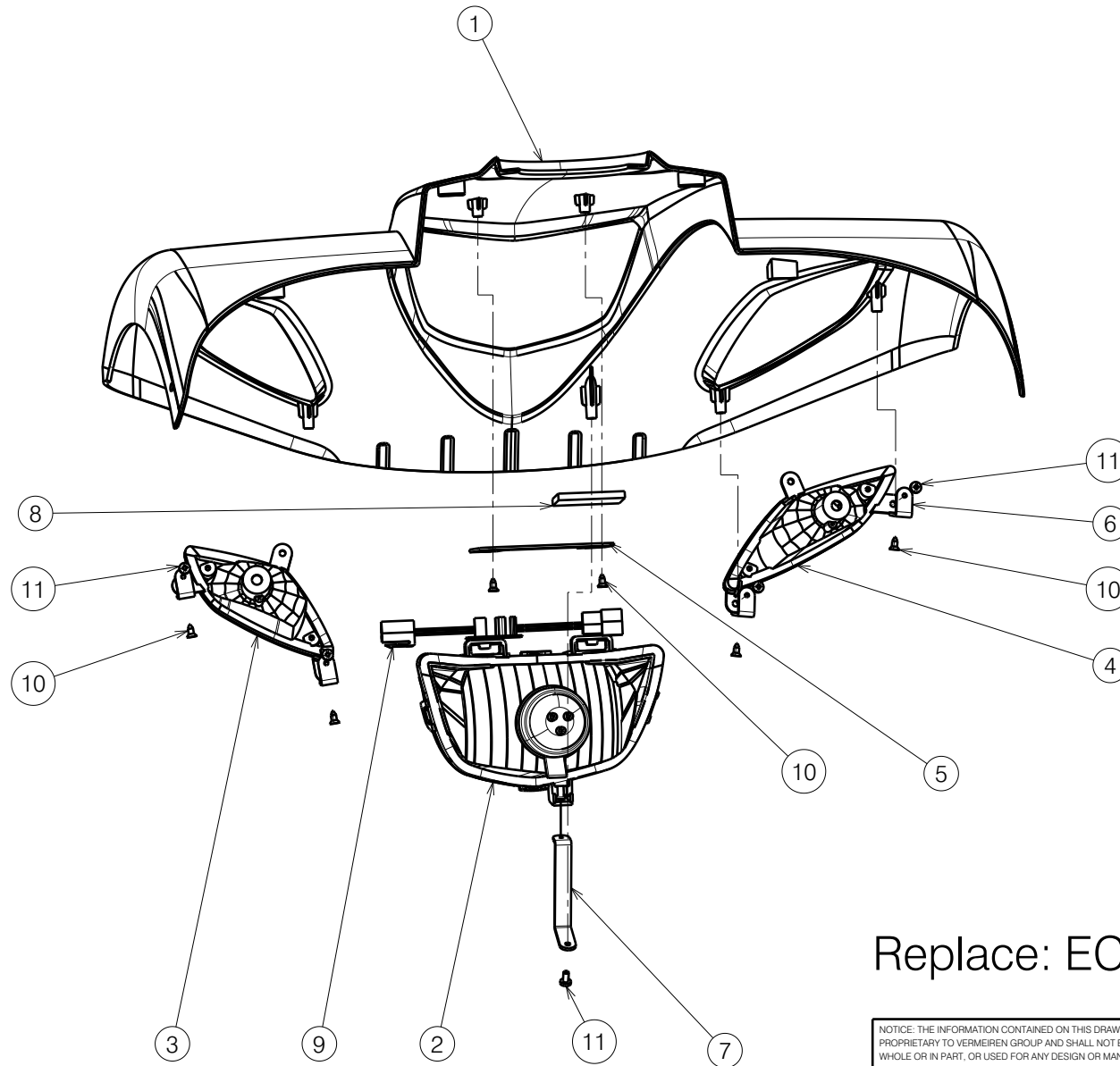

Replace: EC00297

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

SV CARPO SE FRONT SHELL ASM				Plan No. E00063929/A.1
Scale: 1:5	Drafted	Verified	Confirmed	
Date	20/05/2024	20/05/2024	20/05/2024	
Initials	BK	MK	MK	