
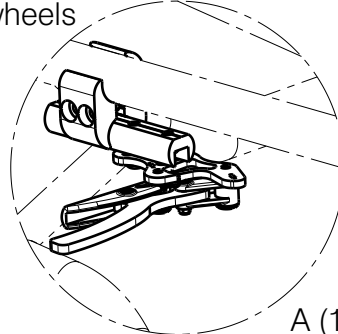



Material			
No.	Plan	Article	Quantity
1	YP06972	2743765	2
2	P00050248	2670465	2
3	XP02231	2993879	2
4	XP02229(R)	2993877	1
	XP02230(L)	2993878	1
5	XP02079	GP15300	4
6	XP02078	GP15226	4
7	XP01914(L)	GP13300	1
	XP01915(R)	GP13400	1
8	XP02922	GP22126	4
9	---	1007521	2
10	M3	2994899	4
11	M6x20 Black	2994010	4
12	M6 DIN 125	2994414	4
13	XP03430	2997530	2
14	M6 DIN 985 Black	2994063	4
15	MBL	2990321	2
16	YP09506	26610...	2
17	M4 DIN985 Black	1907202	4
18	M4 DIN125 Black	1003303	4
19	XP03432	2997532	4
20	XP03431	2997531	4
21	XP00412	2990456	4
22	M4x45 DIN912	2998457	4
23*	P00071746	26852...	2

Quantity per left and right side

\*Pos. 23 for 22 inch wheels

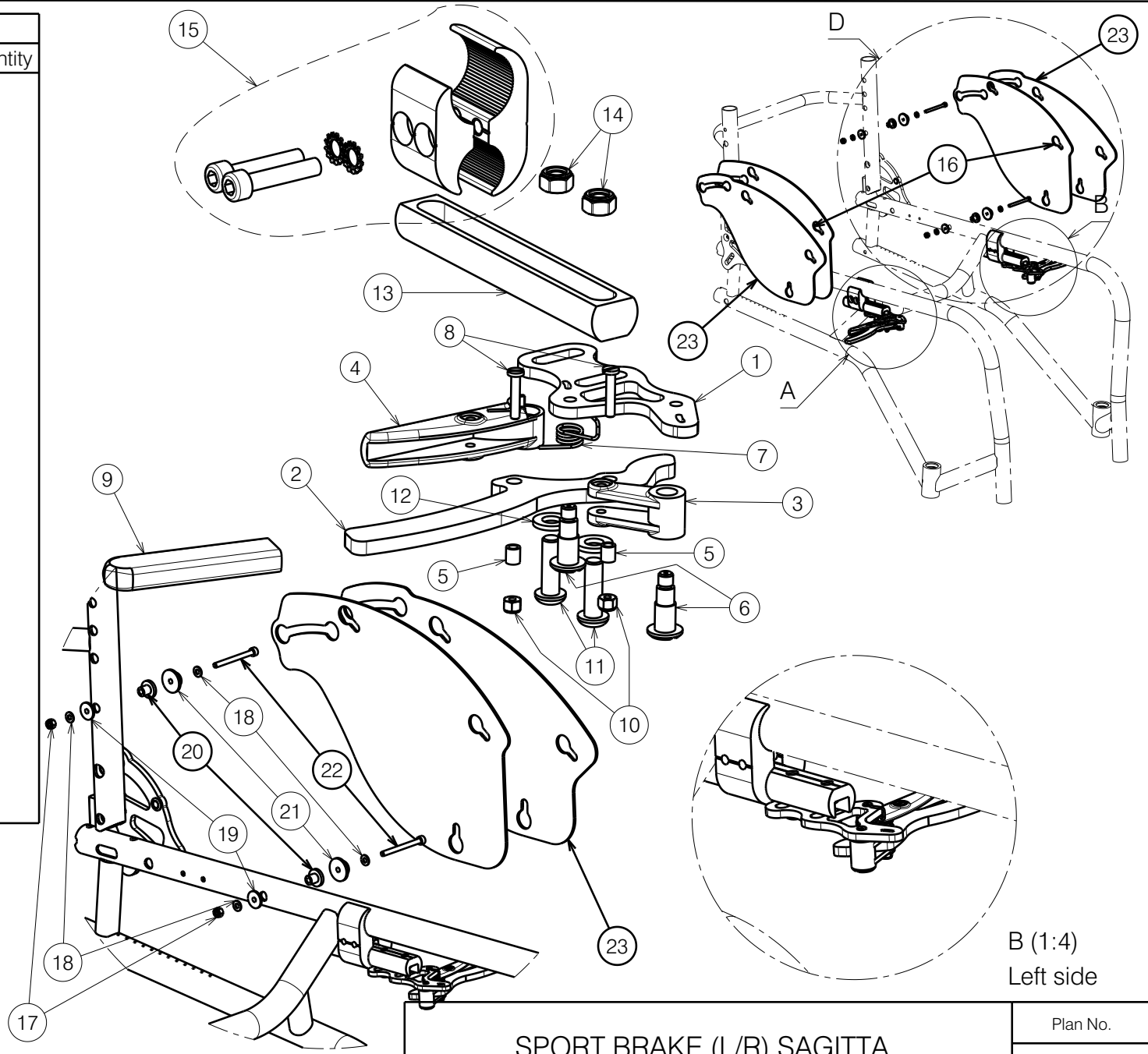

A (1:3)  
Right side

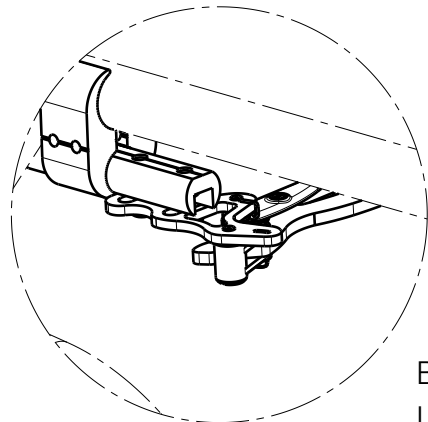

B (1:4)  
Left side

Detail D  
Scale: 1:5

EP00649 / F

SPORT BRAKE (L/R) SAGITTA				Plan No.
				EP00649/ F
Scale: 1:2	Drafted	Verified	Confirmed	
Date	15/11/2022	14/05/2024	14/05/2024	
Initials	WM	WM	AS	