


Material			
	NO.	Plan	Article Number
1	44#		1908967
	46#		1908968
	48#		1908961
	50#		1908962
	55#		1908963
1A	YC00293		1C12454
1B	---		1001680
1C	---		1999977
1D	---		1006999
1E	YC00288(44#)		1C11954
	YC00289(46#)		1C12054
	YC00290(48#)		1C12154
	YC00291(50#)		1C12254
	YC00292(55#)		1C12354
2	YC00294		1C12500
3	---		3990077
4	---		3990078
5	X004820 (Eclips, V200, V300, V300 30°)		1744532 (Ø22.0)
	X004832 (V100, V100D, V200D, V300D)		1744932 (Ø25.4)
6	X004821 (Eclips, V200, V300, V300 30°)		1744632 (Ø22.0)
	X004833 (V100, V100D, V200D, V300D)		1745032 (Ø25.4)
7	X004823		1744832
8	---		1001577
9	X004822		1744732
10	---		1994752
11	---		1001494

EC00055/G

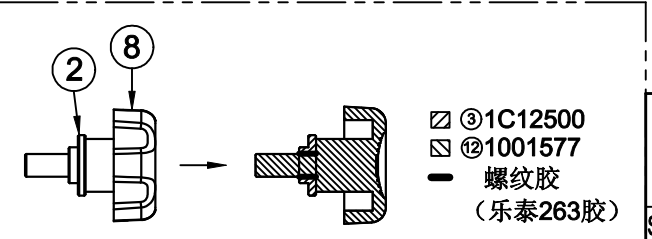

注：③与⑫接触面增加螺纹胶，如上图所示

<b>ANGEL BACKREST FOR ECLIPS / V-SERIES</b>				Plan No. / Plannr. EC00055/G
Scale / Schaal: 1 : 4	Drafted / Getekend	Verified / Nagezien	Confirmed / Bevestigd	
Date / Datum	16/01/2013	16/01/2013	16/01/2013	
Init. / Init.	Lydia	JJQ	TG	